



Table 1: the device sizing chart for single lobe LAA

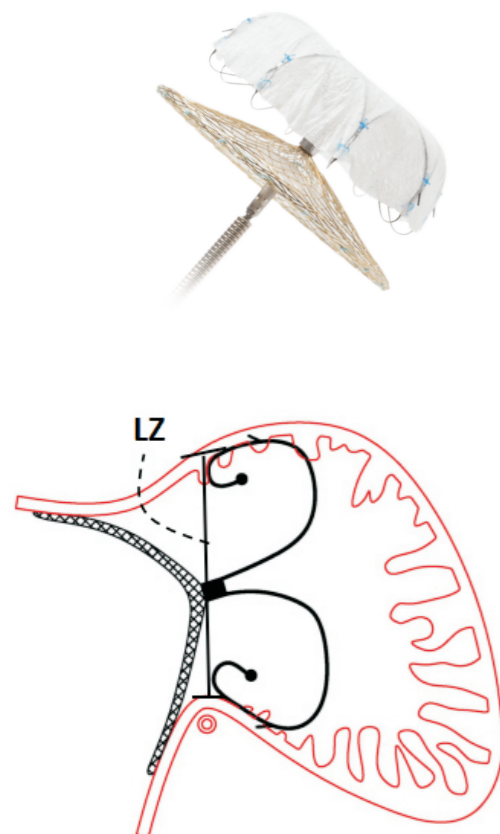

| LZ(mm)    | LAMBRE™ Occluder | D Umbrella Diameter(mm) | D1 Cover Diameter(mm) | LAMBRE™ Delivery System |
|-----------|------------------|-------------------------|-----------------------|-------------------------|
| 11.0~12.5 | LT-LAA-1622      | 16                      | 22                    | 8/9/10Fr; 900mm         |
| 12.5~14.0 | LT-LAA-1824      | 18                      | 24                    |                         |
| 14.0~16.0 | LT-LAA-2026      | 20                      | 26                    | 9/10Fr; 900mm           |
| 16.0~18.0 | LT-LAA-2228      | 22                      | 28                    |                         |
| 18.0~20.0 | LT-LAA-2430      | 24                      | 30                    |                         |
| 20.0~22.0 | LT-LAA-2632      | 26                      | 32                    | 10Fr; 900mm             |
| 22.0~24.0 | LT-LAA-2834      | 28                      | 34                    |                         |
| 24.0~26.0 | LT-LAA-3036      | 30                      | 36                    |                         |
| 24.0~26.0 | LT-LAA-3236      | 32                      | 36                    |                         |
| 26.0~28.0 | LT-LAA-3438      | 34                      | 38                    |                         |
| 28.0~30.0 | LT-LAA-3640      | 36                      | 40                    |                         |

Table 2: the device sizing chart for double or multiple lobes LAA

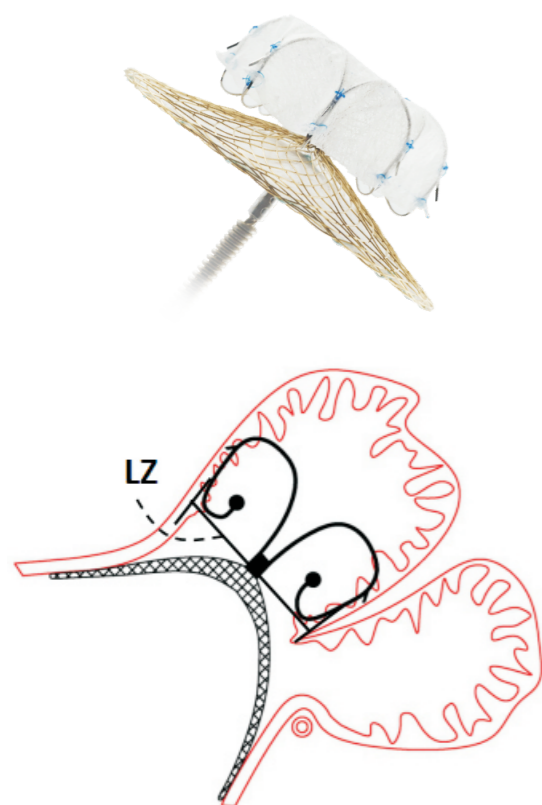

| LZ(mm)    | LAMBRE™ Occluder | D Umbrella Diameter(mm) | D1 Cover Diameter(mm) | LAMBRE™ Delivery System |
|-----------|------------------|-------------------------|-----------------------|-------------------------|
| 11.0~12.5 | LT-LAA-1630      | 16                      | 30                    | 9/10Fr; 900mm           |
| 12.5~14.0 | LT-LAA-1832      | 18                      | 32                    |                         |
| 14.0~16.0 | LT-LAA-2032      | 20                      | 32                    | 10Fr; 900mm             |
| 16.0~18.0 | LT-LAA-2234      | 22                      | 34                    |                         |
| 18.0~20.0 | LT-LAA-2436      | 24                      | 36                    |                         |
| 20.0~22.0 | LT-LAA-2638      | 26                      | 38                    |                         |

Note:

1. LZ: Landing zone
2. Please make sure the depth from the LZ to the distal end of the LAA  $\geq 10$ mm
3. Please make sure the Umbrella is deployed beyond the left circumflex artery

